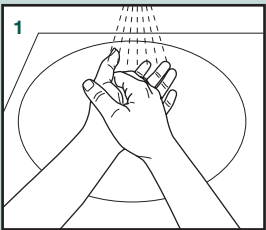
## Deutsch

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## English

Before use, wash hands thoroughly and disinfect them. Avoid touching any part of the catheter surface (except for the funnel) directly. You can use the functional packaging as gripping aid.

Open the gel sachet by folding and squeezing. Make sure the whole content of the sachet is emptied into the packaging.

Spread the gel along the full length of the catheter (except for the funnel).

Clean the opening of the urethra and the surrounding area with a disinfectant suitable for mucous membranes (observing the instructions of the manufacturer of the disinfectant).

Open the urine bag at the front end and release the tip of the catheter. If the catheters sticks to the packaging please pull out twisting it slowly at the same time.

Gently insert the catheter into the urethra until the urine begins to flow. When the urine stops flowing, pull the catheter out slowly.

Push back the catheter into the urine bag and seal this by making a knot. To empty the bag tear at the back end.

Dispose of the catheter according to local authority guidelines.

Should you have any questions or problems, please turn to a medical professional or to our customer adviser.

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